P

Trépano Coronet de Cabo Longo

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Descrição

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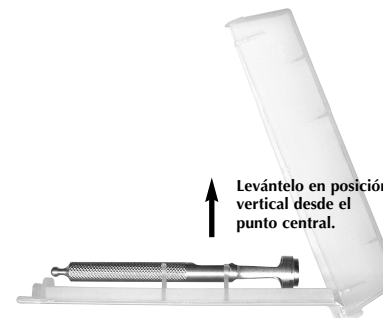
UltraFit™ PK System

Coronet Long Handled Corneal Trephine

Surgical Specialties Corporation

100 Dennis Drive

Reading, PA 19606 U.S.A.

Phone: 610-404-1000

www.angiotech.com

Manufactured For:

Surgical Specialties Corporation

100 Dennis Drive

Reading, PA 19606 U.S.A.

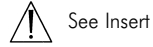
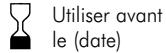
Phone: 610-404-1000

www.angiotech.com

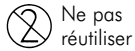
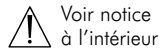
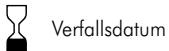
SS0034.2

USA SYMBOLS**LOT** Lot Number**STERILE | R**

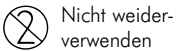
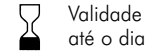
Sterilized by irradiation. Do not use if product is opened or damaged.

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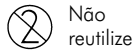
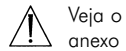
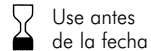
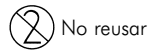
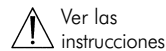
The Coronet long handled trephine is a single use only device with an ultra-sharp cutting edge, supplied with a knurled finger grip for secure manual trephination. The ultra-thin blade profile cuts cleanly through corneal tissue to allow for close approximation between the donor button and the recipient cornea.

Intended Use

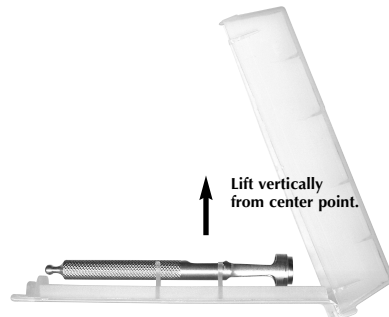
The Coronet long handled trephine is a single use trephine to be used for either trephination of the donor button or trephination of the recipient cornea during corneal transplant surgery.

CAUTION

- The Coronet trephine is supplied ultra-sharp for SINGLE USE ONLY.
- The Coronet trephine is extremely sharp, therefore caution must be exercised at all times when handling and using this trephine.
- To avoid damaging the blade edge, take care when removing the device from the rigid plastic container. Using finger and thumb, grasp the handle between the two holding mounts and lift the trephine vertically.
- Should the device be accidentally dropped onto a hard surface, check the blade integrity under the microscope before use and discard the blade if damaged.

Sterilization**DO NOT RE-STERILIZE.** The trephine is supplied sterile and ready for use.**Disposal**

For safe disposal, return the trephine to the container and secure the lid.

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